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SURFACE 2

Disinfectant and cleaning liquid

cedis Profi SURFACE 2 is a ready-to-use disinfectant for small areas and equipment (e.g. patient chairs, work tables, handpieces and elbows) in the health sector (e.g. doctor's offices, retirement homes, dental and otoplastic laboratories).

Effective against bacteria, viruses (HBV/HIV) and fungi (Candida albicans).

Reaction time: 1 minute

Spray onto and distribute completely on the surfaces. The surfaces must not be rubbed dry during the reaction time.

For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to info@itwindustrialsolutions.com and to the competent public authority of the Member State in which the user and/or patient resides.

Caution:

Harmful to aquatic life with long lasting effects. Contains benzisothiazolinone. May produce an allergic reaction. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use. Avoid release to the environment. Do not use after the expiration date has passed.

Inhaltsstoffe, Ingredients:

Quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl, chlorides 0.25 g, Didecyldimethylammoniumchloride 0.25 g,

Quaternary ammonium compounds, C12-14-alkyl[(ethylphenyl)methyl]dimethyl, chloride 0.25 g / 100 g

500 ml e

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egger Otoplastik + Labortechnik GmbH
ITW LLC & Co. KG